# Compules<sup>®</sup> Tips Gun Applicator Gun

Applikationspistole

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Dispenser a pistola

Pistola de aplicaci3n

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CAUTION: For dental use only.

USA: Rx only.

Content	Page
<b>1 Product description</b> .....	<b>2</b>
<b>2 Safety notes</b> .....	<b>2</b>
<b>3 Step-by-step instructions</b> .....	<b>3</b>
<b>4 Hygiene</b> .....	<b>3</b>
<b>5 Lot number and durability</b> .....	<b>5</b>

## 1 Product description

**Compules® Tips Gun** is an applicator gun for all Dentsply Sirona filling materials delivered in Compules® Tips and Compula® Tips.

## 2 Safety notes

Be aware of the following general safety notes and the special safety notes in other chapters of these Instructions for Use.

### Safety alert symbol.

- This is the safety alert symbol. It is used to alert you to potential personal injury hazards.
- Obey all safety messages that follow this symbol to avoid possible injury.

### 2.1 Precautions

This product is intended to be used only as specifically outlined in these Instructions for Use.

Any use of this product inconsistent with these Instructions for Use is at the discretion and sole responsibility of the dental practitioner.

- Use protective measures for the dental team and patients such as glasses and rubber dam in accordance with local best practice.
- The Compules® Tips Gun is only intended to be used with Dentsply Sirona Compules® or Compula® Tips.
- The material should extrude easily. **DO NOT USE EXCESSIVE FORCE.** Excessive pressure may cause the Compules® Tip to rupture or to eject from the Compules® Tips Gun.
- Please also refer to the Instructions for Use of the products used with this product.

### 3 Step-by-step instructions

#### 3.1 Application

1. Insert Compules® Tip into the notched opening of the Compules® Tips Gun. Be certain that the collar on the Compules® Tip is inserted first.
2. Remove the colored cap from the Compules® Tip. The Compules® Tip may be rotated 360° to gain the proper angle of entrance into the cavity or to the mixing pad.  
**DO NOT USE EXCESSIVE FORCE.**

Indirect use: to prevent Compules® Tip from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the Compules® Tip is handled off-side the dental unit with clean/disinfected gloves. Dispense the necessary amount of material onto a mixing pad and protect against light. Apply material into the cavity with a plastic instrument.

- or -

Direct intraoral use: dispense the material directly into the cavity preparation using a slow, steady pressure.

3. To remove the used Compules® Tip, be sure that the Compules® Tips Gun plunger is pulled back completely by allowing the handle to open to its widest position. Apply a downward motion to the front end of the Compules® Tip and remove.

#### **Excessive force.**

Injury.

1. Apply slow and steady pressure on the Compules® Tips Gun.
2. Do not use excessive force. Compules® Tip rupture or ejection from Compules® Tips Gun may result.

For further information please refer to composite Instruction for Use.

### 4 Hygiene

#### **NOTICE**

#### **Wrong cleaning or disinfection method.**

Damage to the Compules® Tips Gun.

1. Always disassemble Compules® Tips Gun before reprocessing.
2. Do not clean or disinfect with aggressive agents (e.g. solutions based on orange oil or acetone).
3. Start reprocessing within 1 hour after use.

#### **4.1 Disassembly and preparation for reprocessing**

1. Partially close the Compules® Tips Gun and place your thumb on the rear part of the hinge.
2. Push upwards and lift hinge. The Compules® Tips Gun is separated into two parts and the plunger is exposed.
3. Clean Compules® Tips Gun using a soft brush and running cold tap water.
4. Remove restorative material, if any, with a soft paper tissue and alcohol of 70% vol.

#### **4.2 Manual cleaning and disinfection**

1. Discard gloves according to local regulations.
2. Disinfect hands with an appropriate bactericidal, virucidal, and fungicidal hand disinfectant solution according to local regulations. Use according to disinfectant solution manufacturer's Instructions for Use.

3. Use a new clean pair of examination gloves.
4. Clean Compules® Tips Gun with a soft brush under running tap water, and an appropriate cleaning solution (e.g.: alkaline solution: Dürr FD370 [2%]<sup>1</sup>) containing nonionic surfactants (5-15%), chelating agents (< 5%) and adjuvants in an aqueous solution, for at least 30 seconds until free of visible contamination. Pay special attention to device seams and insertions.
5. Remove cleaning solution residue with a cloth soaked with deionized water.
6. Dry device with a lint-free single-use cloth.
7. Thoroughly wipe all surfaces of the Compules® Tips Gun with a single-use cloth in combination with an undiluted, aldehyde-free, alcohol-based, bactericidal, tuberculocidal, virucidal and fungicidal disinfection solution (e.g.: Dürr FD333<sup>1</sup>) approved according to local regulations and use according to disinfectant solution manufacturer's Instructions for Use. Make sure disinfectant solution is compatible with cleaning solution. Pay special attention to device seams and insertions.
8. After 5 minutes remove disinfectant solution residue with a cloth soaked with deionized water.
9. Dry Compules® Tips Gun with a lint-free single-use cloth.

#### 4.3 Automatic Washer-Disinfector

As an alternative to manual cleaning and disinfection, the Compules® Tips Gun can also be processed in a washer-disinfector<sup>2</sup>. Use only properly maintained, calibrated, and approved washer-disinfector according to ISO 15883-1.

1. Place Compules® Tips Gun in the washer-disinfector allowing water and detergent to enter into and drain off from Compules® Tips Gun openings during washer-disinfection program and follow washer-disinfector's Instructions for Use.
2. Run washer-disinfection program with A0 value  $\geq 3000$  (e.g. 5 min at  $\geq 90$  °C) using appropriate detergents (e.g.: neodisher® MediClean [0.5%] (alkaline detergent) and neodisher® Z [0.1%] (acid neutralization and cleaning detergent); both detergents from Dr. Weigert, Hamburg, Germany<sup>3</sup>).

#### 4.4 Autoclaving (optional)

After cleaning and disinfection [4.2 Manual cleaning and disinfection] or after processing in a washer-disinfector [4.3 Automatic washer-disinfector], the Compules® Tips Gun can also be autoclaved.

1. Steam autoclave device unwrapped at 134 °C/ 2 bar = 273 °F/29 psi for 3 minutes 30 seconds.

#### 4.5 Reassembly

1. After reprocessing insert plunger into the Compules® Tips Gun cylinder.
2. Press components together and snap hinge mechanism in place.

Do not continue to use damaged or soiled Compules® Tips Guns.

Compules® Tips Gun can be reprocessed for up to 200 times.

---

<sup>1</sup> Not a registered trademark of Dentsply Sirona, Inc.

<sup>2</sup> For example Miele Washer-Disinfector G7835 CD using program Vario TD.

<sup>3</sup> Not registered trademarks of Dentsply Sirona, Inc.

## **5 Lot number ( LOT ) and durability**

1. Do not use Compules® Tips Gun which is reprocessed for more than 200 times or in use for longer than two years.
2. The following numbers should be quoted in all correspondence:
  - Reorder number
  - Lot number

**© Dentsply Sirona 2016-11-09**

**[These Instructions for Use are based on Master Version 03]**

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Dentsply DeTrey GmbH