Poly-F[®]

Zinc Polycarboxylate Cement

Ciment polycarboxylate de zinc

Instructions for Use _____ English 2

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Poly-F[®]

Zinc Polycarboxylate Cement

CAUTION: For dental use only.

USA: Rx only

Content	Page
1 Product description	2
2 Safety notes	3
3 Step-by-step instructions	4
4 Lot number and expiration date	5

1 Product description

Poly-F[®] is a zinc polycarboxylate cement.

Poly-F[®] cement complies with the requirements of ISO 9917 for zinc polycarboxylate cements.

1.1 Delivery forms

A powder/liquid version for hand mixing.

1.2 Composition

Powder:

- Zinc oxide
- Magnesium hydroxide
- Calcium-alumino-sodium-fluoro-phosphor-silicate glass
- Tin fluoride
- Iron oxide pigments

Liquid:

- Polyacrylic acid
- Water

1.3 Indications

- Cementations of crowns, bridges, inlays, and orthodontic bands.
- Cavity lining/base under restorative materials.
- Intermediate (temporary, less than 1 year) direct restorative filling material.

1.4 Contraindications

- Direct or indirect application to dental pulp (direct or indirect pulp-capping).

2 Safety notes

Be aware of the following general safety notes and the special safety notes in other chapters of these Instructions for Use.

Safety alert symbol.

- This is the safety alert symbol. It is used to alert you to potential personal injury hazards.
- Obey all safety messages that follow this symbol to avoid possible injury.

2.1 Warnings

The material contains polyacrylic acid and may be irritating to skin, eyes and oral mucosa.

- **Avoid eye contact** to prevent irritation and possible corneal damage. In case of contact with eyes rinse with plenty of water and seek medical attention.
- **Avoid skin contact** to prevent irritation. In case of contact, reddish rashes may be seen on the skin. If contact with skin occurs, remove material with cotton and wash thoroughly with soap and water.
- **Avoid contact with oral soft tissues/mucosa** to prevent inflammation. If accidental contact occurs, remove material from the tissues. Flush mucosa with plenty of water and expectorate/evacuate the water. If inflammation of mucosa persists, seek medical attention.

2.2 Precautions

This product is intended to be used only as specifically outlined in these Instructions for Use.

Any use of this product inconsistent with these Instructions for Use is at the discretion and sole responsibility of the dental practitioner.

- Use protective measures for the dental team and patients such as glasses and rubber dam in accordance with local best practice.
- To prevent bottles, powder scoop and mixing pad from exposure to spatter or spray of body fluids or contaminated hands it is recommended that the bottles, powder scoop and mixing pad are handled offside the dental unit with clean/disinfected gloves.

NOTICE

Moisture uptake.

Moisture can adversely affect the properties of the powder.

1. The desiccant crystals integrated in the cap and/or drying capsule protect the powder from absorbing moisture. The color of the desiccant crystals should be orange. The desiccant crystals are no longer effective after the color has turned from orange to colorless.
2. Tightly close bottle immediately after use.

2.3 Adverse reactions

- Eye contact: Irritation and possible corneal damage.
- Skin contact: Irritation. Reddish rashes may be seen on the skin.
- Contact with mucous membranes: Inflammation (see Warnings).

2.4 Storage conditions

Inadequate storage conditions may shorten the shelf life and may lead to malfunction of the product.

- Store at temperatures between 10 °C and 24 °C (50 °F and 75 °F).
- Store in a dry environment (< 80% relative humidity).
- Keep out of direct sunlight.
- Replace cap immediately after use.

3 Step-by-step instructions

3.1 Dentin pre-treatment and pulp protection

1. Cover the dentin close to the pulp with a calcium hydroxide liner (e.g. Dycal®).
2. Rinse and carefully air-dry cavity preparation, but do not desiccate exposed dentin.

3.2 Dosage and mixing

NOTICE

Premature setting of material.

Reduced working time.

1. Keep powder and liquid separated until ready for mixing.

	Cementation	Lining/filling
Mixing ratio	1 scoop powder : 2 drops liquid ¹	2 scoops powder : 2 drops liquid ²
Mixing time	30 seconds	30 seconds
Working time (from start of mixing)	At least 1 minute 45 seconds	At least 1 minute
Setting time (after end of mixing)	2 minutes 30 seconds to 8 minutes	2 minutes to 6 minutes

1. Tap powder bottle on a firm surface to condense powder.
2. Press powder measure firmly into the powder.
3. Strike off excess powder with a spatula to leave a flat surface level with the edge of the measure.
4. Place powder onto a mixing pad or glass slab by tapping the end of measure with the spatula ³.
5. Incline bottle at a vertical angle and squeeze it to produce the number of required drops.
6. Release pressure on the bottle after each drop is dispensed.
7. Using a suitable spatula, incorporate all the powder rapidly into the liquid.
8. Mix for 30 seconds until a creamy consistency is reached for a cementation material, or a putty-like consistency for a lining/filling material.

¹ The same powder : liquid ratio (5 g : 1 g) is proposed for testing purposes according to ISO 9917.

² The same powder : liquid ratio (10 g : 1 g) is proposed for testing purposes according to ISO 9917.

³ Mixing and application of the material should be done at normal room temperature. Higher temperatures shorten the working and setting times of the cement. Under warm conditions or if a long working time is required, it is therefore recommended that the glass slab be cooled. Testing to ISO 9917 is carried out at 23 °C ± 1 °C and a relative humidity of 50 ± 10%.

3.3 Placement

1. Apply cement immediately after mixing whilst appearance is glossy. Do not use after it starts to “cobweb”.
2. Remove excess material at once or when the cement has completely hardened. Do not touch the material when it is in the intermediate rubbery condition.

3.4 Finishing

1. Excess material may be removed at once or after completion of setting time (see 3.2 Dosage and mixing). Do not touch the cement when it is in the intermediate, rubbery condition. When used as a temporary restoration, finishing burs are recommended. For removal of surplus material after crown cementation, appropriate hand instruments, like a dental probe, and dental floss are used.

3.5 Cleaning instruments

Zinc polycarboxylate cements adhere to instruments and glass slabs.

1. Immediately after use, either clean instruments with water or drop in disinfectant or water to facilitate subsequent cleaning.

4 Lot number () and expiration date ()

1. Do not use after expiration date. ISO standard is used: “YYYY-MM”
2. The following numbers should be quoted in all correspondence:
 - Reorder Number
 - Lot number
 - Expiration date

© Dentsply Sirona 2014-07-31

[These Instructions for Use are based on Master Version 03]

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Dentsply DeTrey GmbH