SmartLite® PS

**Pen-Style,
High-Power LED
Curing Light**

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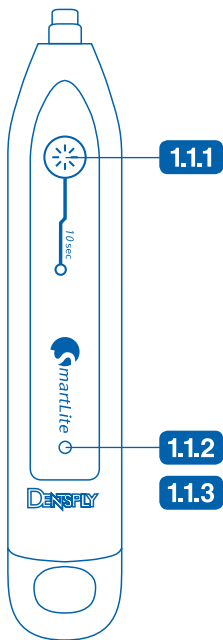
For better dentistry

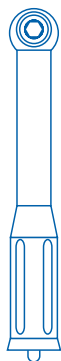
DENTSPLY

SmartLite® PS curing light – operation at a glance

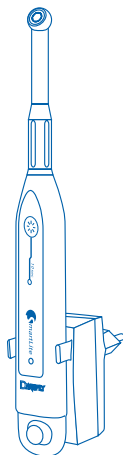
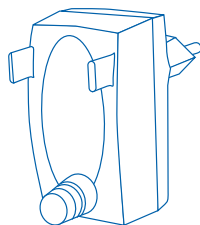
1

2

3

Instructions for Use	2
Gebrauchsanweisung	12
Mode d'emploi	22
Istruzioni per l'uso	32
Instrucciones de uso	42
Инструкция по применению	52
Instruções de utilização	62
Bruksanvisning	72
Gebruiksaanwijzing	82
Instrukcja użytkowania	92
Brugsanvisning	102
Οδηγίες χρήσης	112
Návod k použití	122
Návod na použitie	132
Használati utasítás	142
Kullanım Talimatı	152

Pen-Style, High-Power LED Curing Light

CAUTION: For dental use only.
USA: Rx only.

Content	Page
1 Product description.....	2
2 Safety notes	4
3 Step-by-step instructions	6
4 Hygiene.....	7
5 Maintenance.....	8
6 Reorder information, technical data, warranty terms.....	9

1 Product description

The **SmartLite® PS** curing light is a cordless pen-style, LED light polymerization device for use in dentistry.

SmartLite® PS curing light is characterized by:

- Small size and low weight.
- Cordless operation.
- Powerful light emitting diode (LED) of latest technology, placed at tip head, acting directly on the material to be cured.
- Polymerization area (optic effective cross-sectional area) of minimum 8.0 mm in diameter.
- Freely adjustable LED tip, rotating by 360°.
- 15° angulation of light probe for optimum access.
- Standard 10-second curing cycle.

1.1 SmartLite® PS curing light – operation at a glance

1.1.1 ON/OFF key

Activates device and starts or disrupts the 10-second curing cycle.

1.1.2 Battery light

Continuously green (only indicated when SmartLite® PS curing light handpiece is inserted into the charger): SmartLite® PS curing light is fully recharged and ready for operation.

Flashing green (only indicated when SmartLite® PS curing light handpiece is inserted into the charger): SmartLite® PS curing light is still being recharged.

Fast flashing red: Approximately 2 minutes curing time left – SmartLite® PS curing light needs recharging.

Slowly flashing red: SmartLite® PS curing light is fully discharged (no curing time left) and needs recharging.

1.1.3 Audible signals

Short blip: Start and end of curing cycle.

Repetitive short blip approximately every 40 seconds (battery light flashes red): SmartLite® PS curing light needs recharging.

Blip followed by 4-tone melody: LED tip is not attached to handpiece or malfunction of device: Do not use device.

1.2 Indications

- Intra-oral light-curing of direct restorative materials, adhesives, and sealants which cure at a wavelength of 460 - 490 nm.

1.3 Contraindications

- Luting of indirect restorations.
- Use in patients prone to photobiological reactions (including patients with solar urticaria or erythropoietic protoporphyria) or those currently undergoing treatment with photosensitizing pharmaceuticals.

2 Safety notes

Be aware of the following general safety notes and the special safety notes in other chapters of these Instructions for Use.

Safety alert symbol.

- This is the safety alert symbol. It is used to alert you to potential personal injury hazards.
- Obey all safety messages that follow this symbol to avoid possible injury.

2.1 Warnings

2.1.1 SmartLite® PS

- Do not use the device as a tissue retractor, as this may damage the connection between LED tip and handpiece.
- Do not use a damaged device, e.g. if among others the glass cover of the LED tip is scratched, broken or missing.
- Do not use device when malfunction is indicated by audible signal blip followed by 4-tone melody.
- The SmartLite® PS Eye Protection Shields will wear over time. Exchange defect shield against spare shields included in the package or available as refill (see also 6.1 Accessories).

2.1.2 Charger

- High voltages are present inside the charger. Do not open charger or handpiece. Use only under dry conditions. Do not use if the charger is wet. Only authorized technicians should repair the LED tip, handpiece or charger.
- Do not use for voltages different from the range indicated on the charger.
- Note: To disconnect the power supply, the charger must be unplugged.
- Always unplug the charger before reprocessing.

2.2 Precautions

This product is intended to be used only as specifically outlined in these Instructions for Use.

Any use of this product inconsistent with these Instructions for Use is at the discretion and sole responsibility of the dental practitioner.

- Anyone with a history of retinopathy should consult their eye specialist before operating this unit. Use the SmartLite® PS curing light extremely carefully and comply with all the necessary safety precautions (including wearing suitable, light filtering safety glasses).
- Anyone who has had a cataract operation may be especially sensitive to light and should be advised against undergoing treatment with a SmartLite® PS curing light unless adequate safety precautions are taken such as wearing suitable, light filtering safety glasses.
- Do not use SmartLite® PS curing light which has not been properly reprocessed. Protect SmartLite® PS curing light from contamination by applying the single use Disposa-Shield® protective barrier. Disposa-Shield® protective barriers are intended for single use only. Discard after use. Do not reuse in other patients in order to prevent cross-contamination.

- For electrical medical devices specific safety precautions apply which have to be observed¹. Do not use mobile phones or other portable communication tools, as they can interfere with the function of SmartLite® PS curing light.
- Never aim the light directly at unprotected soft tissues, as this may cause injury or irritation. Do not aim the light at eyes. Light reflected from the tooth surface may also injure eyes. Use the protective shield supplied with the unit or suitable, light filtering safety glasses.
- The time intervals between the curing of consecutive layers have to be at least 30 seconds. If time intervals are shorter, this will result in heat build-up of the LED tip which may cause discomfort or tissue irritation.
- Limit the action of the light to the area being treated.

¹ For technical details (on electromagnetic emissions and electromagnetic immunity) see: www.dentsply.eu – chapter: IFU/MSDS.

3 Step-by-step instructions

3.1 Installation and charging

1. Plug in charger.
2. Insert the handpiece **1** into the charger **3**. The battery light will be flashing green, indicating battery is being recharged.
3. Charge the device for at least 10 hours prior to first use. The battery light will be continuously green, indicating battery is fully recharged.

A new completely charged device allows approximately 150 ten-second cures (total operating time approximately 25 minutes). Thus, under most conditions of use, the device will normally need recharging at the end of each day.

3.2 Operation

1. Attach the LED tip **2** to the handpiece **1**. The device is now ready for operation.
2. Protect LED tip from contamination by applying the single use Disposa-Shield® protective barrier.
3. Use the protective shield supplied with the unit or suitable, light filtering safety glasses.
4. Adjust the LED tip: The LED tip rotates by 360°, thus the position of light emission may be freely adjusted. The LED tip must be adjusted to an angle that allows optimal curing of the restoration. The LED tip should be positioned **as closely as possible to the restoration** and may even touch the restoration as the LED tip is protected by the Disposa-Shield® protective barrier.
5. Polymerization: Briefly press the ON/OFF key **1.1.1**. Start and end of the 10-second curing cycle are indicated by audible signals. Curing times for a range of materials are outlined in the Curing Guide (Guides) provided. To cure a restorative material that requires a curing time longer than 10 seconds, repeat activation by pressing the ON/OFF key after the end of each 10-second curing cycle.

4 Hygiene

4.1 Disposa-Shield® protective barrier

The use of the Disposa-Shield® protective barrier is a mandatory precautionary measure against contamination, but does not substitute reprocessing of the SmartLite® PS curing light.

1. Remove Disposa-Shield® protective barrier immediately after use.

Cross-contamination.

Infection.

1. Disposa-Shield® protective barrier shields are intended for single use only. Do not reuse in order to prevent cross-contamination or malfunction.
2. Discard in accordance with local regulations.

4.2 Charger, handpiece, LED tip

Electrical short-circuit or dangerous malfunction.

Injury.

1. Disconnect the charger from the power source during reprocessing.
2. Safeguard charger, handpiece and LED tip against liquid penetration during cleaning and disinfection.

4.2.1 Cleaning

1. Clean item using a moist cloth with water and detergent until the item is free of visible contamination.

4.2.2 Disinfection

1. Disinfect item using a moist cloth with an alcohol-based surface disinfectant (e.g. Dürr FD 322, Ecolab Incidin Liquid, S&M Mikrozid AF Liquid, B. Braun Meliseptol rapid) according to national/local regulations.
2. Follow the disinfectant manufacturer's Instructions for Use.

Items can be reprocessed for up to 3,600 times.

NOTICE

Wrong cleaning method.

Damage to SmartLite® PS curing light.

1. Do not submerge into disinfection solutions.
2. Do not clean with highly aggressive disinfecting solutions (e.g. solutions based on orange oil or acetone).

4.3 SmartLite® PS Eye Protection Shield

4.3.1 Cleaning

1. Clean item using a moist cloth with water and detergent until the item is free of visible contamination.

4.3.2 Disinfection

1. Disinfect item using a moist cloth with a water-based hospital-level disinfection solution according to national/local regulations. Agents containing organic solvents may tend to dissolve the plastic.
2. Follow the disinfectant manufacturer's Instructions for Use.

SmartLite® PS Eye Protection Shield can be reprocessed for up to 100 times.

NOTICE

Wrong cleaning method.

Damage to SmartLite® PS Eye Protection Shield.

1. Do not clean with highly aggressive disinfecting solutions (e.g. solutions based on orange oil or acetone).

5 Maintenance

5.1 Light output control

1. Make sure that the LED aperture is clean and scratch-free, otherwise light output will be reduced and may be insufficient for proper curing of the material.
2. Regular control of light efficiency is recommended (e.g. by using CureRite™ light intensity testing device). The light intensity measured by the CureRite™ light intensity testing device should exceed 730 mW/cm².

5.2 Battery

- When the battery light flashes **green**, the battery is charging. Upon complete recharging, the battery light remains continuously green. The battery needs approximately 10 hours to be fully recharged. A fully charged, new battery has a capacity of approximately 150 ten-second cures (approximately 25 minutes operating time). Therefore, under most conditions of use, the device only needs to be recharged overnight.
- When the battery light flashes **red (fast)**, the battery needs to be recharged. Approximately another 2 minutes curing time remains for completion of the treatment. Light output is not reduced during this period. Approximately after these 2 minutes, the device switches off automatically and continues to flash **red (slowly)** indicating the need to recharge.
- The maximum capacity of the rechargeable battery will be reached after initial charging cycles.
- Discharge the device by regular use at least every 6 months and recharge the battery immediately thereafter.
- If the SmartLite® PS curing light is not used during an extended period (e.g. holidays) leave the handpiece in the operable charger. Otherwise the batteries may become severely discharged, which may cause irreversible damage to the battery.

6 Reorder information, technical data, warranty terms

6.1 Accessories

Accessory

	Reorder no.
SmartLite® PS LED tip	644.40.050
SmartLite® PS Eye Protection Shield	644.40.051
SmartLite® PS (230V, EU, Model no. 100EU) Charger	644.40.015
SmartLite® PS (120V, US, Model no. 101US) Charger	644.40.025
SmartLite® PS (230V, UK, Model no. 102UK) Charger	644.40.035
SmartLite® PS (100V, JPN, Model no. 103JPN) Charger	644.40.045
Disposa-Shield®	A880065

6.2 Serial number

The serial number  should be quoted in all correspondence, which requires identification of the product.

6.3 Specifications and equipment class

Environmental Note

Battery recycling:

This device is provided with a nickel metal hydride battery. For environmental reasons, please dispose of the device according to local environmental guidelines or regulations.

AC supply connection:

EU charger (Model no. 100EU):
220 V...240 V ~ 50 Hz...60 Hz 2.5 W
US charger (Model no. 101US):
110 V...130 V ~ 50 Hz...60 Hz 2.5 W
UK charger (Model no. 102UK):
220 V...240 V ~ 50 Hz...60 Hz 2.5 W
JPN charger (Model no 103JPN):
100 V~, 50/60 Hz, 60 mA max.

Power input:

EU charger (Model no. 100EU): 2.5 W
US charger (Model no. 101US): 2.5 W
UK charger (Model no. 102UK): 2.5 W
JPN charger (Model no 103JPN): 60 mA max.

Equipment class:

Class II

Protection from electric shock:

Type BF 

Protection from ingress of liquids:

IP X4

Operation:

For permanent operation mode:
10 seconds ON / 30 seconds OFF
Ambient temperature:
Between -10 °C and +32 °C
Relative humidity:
Between 30% and 100%

Storage:	Ambient temperature: Between -10 °C and +35 °C (14 °F and 95 °F) Relative humidity: Between 30% and 80% (non-condensing)
Transport:	Ambient temperature: Between -10 °C and +60 °C (14 °F and 140 °F) Relative humidity: Between 30% and 80% (non-condensing) Atmospheric pressure: Between 500 hPa and 1060 hPa
Battery performance:	A new handpiece will operate approximately 25 minutes starting upon complete recharging. Time for battery recharge: Approximately 10 hours.
Battery over current protection:	Resettable fuse
LED tip with serial number format:	XY 00000
Light emitting diode:	5 W LED
Average light intensity:	Approximately 950 mW/cm ² (CureRite™ output)
Output peak wavelength range:	Between 450 nm and 490 nm (intensity maximum at 460 nm)
LED tip with serial number format:	P-XY abcde
Light emitting diode:	3 W LED
Average light intensity:	Approximately 1200 mW/cm ² (CureRite™ output)
Output peak wavelength range:	Between 460 nm and 490 nm (intensity maximum at 475 nm)
Unit handpiece dimensions:	Length 159 mm Width 28 mm
Tip dimensions:	Length 93 mm Width 14 mm
Unit weight:	Handpiece and tip 100 grams Charger 45 grams

Complies with the following Directives/Standards:

93/42/EEC	Council Directive 93/42/EEC of 14 June 1993 concerning medical devices
EN 60601-1	1990 – Medical Electrical Equipment (General requirements for safety)
EN 60601-1 A1	1993 – Amendment 1 to EN 60601-1
EN 60601-1 A2	1995 – Amendment 2 to EN 60601-1
EN 60601-1-2	1993 – Medical Electrical Equipment Part 1: General requirements for safety 2: Collateral standard: Electromagnetic compatibility – Requirements and tests

The US version of SmartLite® PS curing light (Model no. 101US and the respective charger) complies with:

MEDICAL EQUIPMENT
WITH RESPECT TO ELECTRIC SHOCK,
FIRE AND MECHANICAL HAZARDS ONLY
IN ACCORDANCE WITH UL60601-1 AND CAN/CSA C22.2
No.601.1
23GC

6.4 Warranty terms

DENTSPLY grants a 2-year warranty on all components of the SmartLite® PS except the battery. The battery is covered by a 1-year warranty. The warranty commences on the date of purchase. Within the warranty period, DENTSPLY will eliminate free of charge any defects in the appliance resulting from faults in material or workmanship either by repairing or exchanging parts or exchanging the whole device at DENTSPLY's discretion.

Not covered by this warranty: Damage arising from improper use (operation with incorrect current/voltage, unsuitable power point, breakage, cleaning by other than the recommended methods), normal wear and defects which have a negligible effect on the value or operation of the appliance.

This warranty becomes void if repairs are undertaken by unauthorized persons.

This warranty extends to every country where this device is supplied by DENTSPLY or its appointed distributor and where no import restrictions or legal regulations hinder or prevent service being given under warranty.

Service under this warranty does not affect the expiration date of the warranty. The warranty on parts or entire devices which are exchanged ends when the warranty on this device expires.

In the event of a claim of this device, return the complete device (charging unit and the LED curing light) together with the invoice to your dealer or send it to your nearest DENTSPLY Service Center.

All other claims including those for damages resulting from this warranty are excluded unless our liability is legally mandatory.

© DENTSPLY DeTREY 2012-02-03

[These Instructions for Use are based on Master Version 03]

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DENTSPLY Italia S.r.l.
DeguDent Austria GmbH
DENTSPLY DeTrey GmbH