# Spectrum<sup>®</sup>

## Universal Microhybrid Composite Restorative

Composite micro-hybride de restauration universel

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CAUTION: For dental use only.

USA: Rx only.

Content	Page
<b>1 Product description</b> .....	<b>2</b>
<b>2 Safety notes</b> .....	<b>3</b>
<b>3 Step-by-step instructions</b> .....	<b>5</b>
<b>4 Hygiene</b> .....	<b>7</b>
<b>5 Lot number and expiration date</b> .....	<b>7</b>

### 1 Product description

**Spectrum<sup>®</sup>** restorative is a visible-light activated, radiopaque microhybrid composite for anterior and posterior restorations. The composite is delivered in traditional syringes and is available in a selection of precise VITA<sup>®1</sup> shades.

#### 1.1 Delivery forms

- Syringes

#### 1.2 Composition

- TEGDMA
- Dimethacrylate
- Multifunctional polymethacrylate
- Camphorquinone
- Ethyl-4(dimethylamino)benzoate
- Butylated hydroxy toluene (BHT)
- UV stabilizer
- Barium-aluminium-borosilicate glass (mean particle size < 1 µm)
- Silanated barium-boron-fluoro-alumino-silicate glass (mean particle size < 1 µm)
- Highly dispersed silicon dioxide (particle size 10-20 nm)
- Fluorescent agent
- Iron oxide pigments
- Titanium dioxide

#### 1.3 Indications

- Restorations of all cavity classes in anterior and posterior teeth.

#### 1.4 Contraindications

- Use with patients who have a history of severe allergic reaction to methacrylate resins or any of the other components.

<sup>1</sup> Not a registered trademark of Dentsply Sirona, Inc.

## 1.5 Compatible adhesives

The material is to be used following application of a compatible (meth)acrylate-based dentin/enamel adhesive such as Dentsply Sirona adhesives designed for use with visible light cured composite restoratives (e.g. the Spectrum® bond, prime&bond® one Etch&Rinse or prime&bond® one Select).

## 2 Safety notes

Be aware of the following general safety notes and the special safety notes in other chapters of these Instructions for Use.

### Safety alert symbol.

- This is the safety alert symbol. It is used to alert you to potential personal injury hazards.
- Obey all safety messages that follow this symbol to avoid possible injury.

### 2.1 Warnings

The material contains methacrylates and polymerizable monomers which may be irritating to skin, eyes and oral mucosa and may cause allergic contact dermatitis in susceptible persons.

- **Avoid eye contact** to prevent irritation and possible corneal damage. In case of contact with eyes rinse with plenty of water and seek medical attention.
- **Avoid skin contact** to prevent irritation and possible allergic response. In case of contact, reddish rashes may be seen on the skin. If contact with skin occurs, remove material with cotton and alcohol and wash thoroughly with soap and water. In case of skin sensitization or rash, discontinue use and seek medical attention.
- **Avoid contact with oral soft tissues/mucosa** to prevent inflammation. If accidental contact occurs, remove material from the tissues. Flush mucosa with plenty of water and expectorate/evacuate the water. If inflammation of mucosa persists, seek medical attention.

### 2.2 Precautions

This product is intended to be used only as specifically outlined in these Instructions for Use.

Any use of this product inconsistent with these Instructions for Use is at the discretion and sole responsibility of the dental practitioner.

- If refrigerated, allow material to reach room temperature prior to use.
- Use protective measures for the dental team and patients such as glasses and rubber dam in accordance with local best practice.
- Contact with saliva, blood and sulcus fluid during application may cause failure of the restoration. Use adequate isolation such as rubber dam.
- Tightly close syringes immediately after use.
- The syringes cannot be reprocessed. To prevent syringes from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the syringe is handled offside the dental unit with clean/disinfected gloves. Discard syringe if contaminated.
- The shade guide and the individual shade guide tabs cannot be reprocessed. To prevent the shade guide and the individual shade guide tabs from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the shade guide and the individual shade guide tabs are handled with clean/disinfected gloves.

- Interactions:
  - Do not use eugenol- or hydrogen peroxide-containing materials in conjunction with this product since they may interfere with hardening of the product.

### **2.3 Adverse reactions**

- Eye contact: Irritation and possible corneal damage.
- Skin contact: Irritation or possible allergic response. Reddish rashes may be seen on the skin.
- Contact with mucous membranes: Inflammation (see Warnings).

### **2.4 Storage conditions**

Inadequate storage conditions may shorten the shelf life and may lead to malfunction of the product.

- Store at temperatures between 2 °C and 28 °C (35 °F and 82 °F).
- Keep out of direct sunlight and protect from moisture.
- Do not freeze.
- Do not use after expiration date.
- Store in a dry environment (< 80% relative humidity).

### 3 Step-by-step instructions

#### 3.1 Shade selection

Shade selection should be made prior to the restorative procedure while the teeth are hydrated. Remove extraneous plaque or surface stain if any with prophylaxis paste. Use the Spectrum® shade guide provided with samples of original Spectrum® restorative material. The color coding dot on the shade guide matches the colored label on the syringe.

Alternatively, a VITAPAN®<sup>2</sup> classical shade guide may be used. The Spectrum® shade corresponds to the central part of the respective VITA® tooth.

#### **Avoid contamination of the shade guide.**

To prevent the shade guide from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the shade guide is handled offside the dental unit with clean/disinfected gloves. For shade selection, remove individual tabs from shade guide holder.

#### 3.2 Cavity preparation

1. Clean the tooth surface with a fluoride free prophylaxis paste (e.g. Nupro® prophylaxis paste).
2. Prepare cavity (unless not necessary, e.g. cervical lesion).
3. Use adequate isolation such as rubber dam.
4. Rinse surface with water spray and carefully remove rinsing water. Do not desiccate the tooth structure.

#### 3.2.1 Matrix placement and wedging (Class II restorations)

1. Place a matrix (e.g. AutoMatrix® or Palodent® matrix system) and wedge. Burnishing of the matrix band will improve contact and contour. Pre-wedging or BiTine® ring placement is recommended.

#### 3.3 Pulp protection, tooth conditioning/dentin pre-treatment, adhesive application

Refer to adhesive manufacturer's Instructions for Use for pulp protection, tooth conditioning and/or adhesive application. Once the surfaces have been properly treated, they must be kept uncontaminated. Proceed immediately to placement of the material.

#### 3.4 Application

Apply material in increments immediately after the application of the adhesive. The material is designed to be cured in increments up to a 3 mm depth/thickness.

#### **Avoid contamination of the syringe.**

To prevent the syringe from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the syringe is handled offside the dental unit with clean/disinfected gloves.

1. Remove the cap.
2. Turn the handle of the syringe slowly in a clockwise direction and dispense the necessary amount of the material onto a mixing pad.
3. Point the front tip of the syringe upwards and turn the handle anti-clockwise to prevent oozing of the material.

<sup>2</sup> Not a registered trademark of Dentsply Sirona, Inc.

4. Immediately close the syringe with the cap.
5. Protect the restorative material on the mixing pad against light.

### 3.5 Light curing

1. Light cure each area of the restoration surface with a suitable curing light.<sup>3</sup> Minimum light output must be at least 500 mW/cm<sup>2</sup>. Refer to curing light manufacturer's Instructions for Use for compatibility and curing recommendations.
2. Cure each layer according to the table below.

	<p><b>Insufficient curing.</b></p> <p>Inadequate polymerization.</p> <ol style="list-style-type: none"> <li>1. Check compatibility of curing light.</li> <li>2. Check curing cycle.</li> <li>3. Check minimum irradiance.</li> <li>4. Cure each area of each increment for the recommended curing time.</li> </ol>
---	--

#### Curing Light Output $\geq$ 500 mW/cm<sup>2</sup>

Shades	Increment/layer	Curing time
A1; A2; A3; A3,5; B1; Incisal-B1; B2; C2; C3; D3	3 mm	20 sec
Opaque-A2; Opaque-A3,5; A4; B3; C4	2 mm	20 sec

### 3.6 Finishing and polishing

1. Contour the restoration using finishing burs or diamonds.
2. Use Enhance<sup>®</sup> finishing devices for additional finishing.
3. Prior to polishing, check that surface is smooth and free of defects.
4. For achieving a very high luster of the restoration, we recommend ceram.x<sup>®</sup> gloss, Enhance<sup>®</sup> and PoGo<sup>®</sup>, finishing and polishing instruments, Prisma<sup>®</sup> Gloss™ Regular and Extrafine polishing paste.

For contouring, finishing, and/or polishing, follow the manufacturer's Instructions for Use.

<sup>3</sup> Curing light designed to cure materials containing camphorquinone (CQ) initiator. Peak of spectrum in the range of 440-480 nm.

## 4 Hygiene

### 4.1 Syringes – cross-contamination

#### **Cross-contamination.**

Infection.

1. The syringes cannot be reprocessed.
2. To prevent the syringes from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the syringes are handled with clean/disinfected gloves. Do not reuse syringes if contaminated.
3. Dispose contaminated syringes in accordance with local regulations.

### 4.2 Shade guide and individual shade guide tabs – cross-contamination

#### **Cross-contamination.**

Infection.

1. The shade guide and the individual shade guide tabs cannot be reprocessed.
2. To prevent the shade guide and the individual shade guide tabs from exposure to spatter or spray of body fluids or contaminated hands it is mandatory that the shade guide and the individual shade guide tabs are handled with clean/disinfected gloves. Do not reuse shade guide and the individual shade guide tabs if contaminated.
3. Dispose contaminated shade guide and the individual shade guide tabs in accordance with local regulations.

## 5 Lot number ( ) and expiration date ( )

1. Do not use after expiration date.  
ISO standard is used: “YYYY-MM” or “YYYY-MM-DD”.
2. The following numbers should be quoted in all correspondence:
  - Reorder number
  - Lot number
  - Expiration date

© Dentsply Sirona 2016-04-22

[These Instructions for Use are based on Master Version 02]

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Dentsply DeTrey GmbH